Quantitative Fit Test Adapter

Instructions for use of the 3M™ Quantitative Fit Test Adapter 601

The 3M™ Quantitative Fit Test Adapter 601 is designed to permit quantitative fit testing with an individual's personal 3M™ 6000, 7000, FF-400, or 3M™ Scott™ AV-3000 Series Facepiece with bayonet style cartridge/filter attachments. This is an alternate method to sharing a probed fit test respirator.

IMPORTANT: Keep these *Replacement Instructions* for reference.

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98-0060-0176-6
34-8726-3851-4

3M™ Quantitative Fit Test Adapter 601 Installation

1. Remove the filter from the right side of the respirator (Fig. 1).
2. Cut a length of sampling tubing as follows;
 - 3M™ Half Facepiece 6000 Series - 3 inches
 - 3M™ Full Facepiece 6000 Series - 4 1/4 inches
 - 3M™ Half Facepiece 7500 Series - 3 inches
 - 3M™ Full Facepiece 7800 Series - 4 1/4 inches
 - 3M™ Full Facepiece FF-400 Series - 4 1/4 inches
 - 3M™ Scott™ AV-3000 Series Full Facepiece - 6 inches
3. Make certain the 601 Adapter's inhalation valve is in place (Fig. 2).
4. Slide one end of cut sampling tube onto the inside fitting of the 601 Adapter (Fig. 3).
5. Insert the other end of the sampling tube through the respirator filter holder and past the respirator's inhalation valve. Inserting the sample tube through the filter holder in the area shown minimizes the chance of kinking tubing during adapter installation. With the full facepiece respirator, feed this tubing end also past the nose cup (and in the case of an AV-series full facepiece) the AV-632 Bayonet Adapter inhalation valve (Fig. 4).
6. Engage the 601 Adapter to the respirator by aligning parts, pushing together and turning adapter clockwise 1/4 turn to stop.
7. Attach a suction cup to the forked end of a retainer clip (Fig. 5).
8. Install the retainer clip and suction cup onto the sampling tube inside the respirator facepiece (Fig. 6).
9. Fasten the suction cup to the inside surface of the respirator facepiece or full facepiece nose cup. The suction cup should be located between the nose and mouth of the test subject. It may be necessary to moisten the suction cup with a drop of water to make it

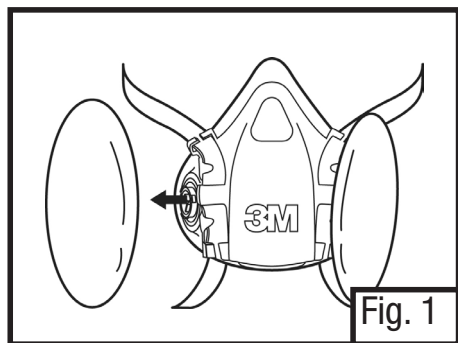
Fig. 1

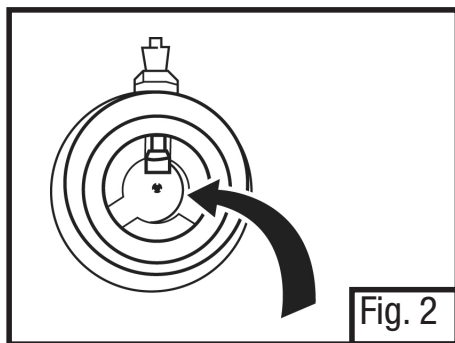
Fig. 2

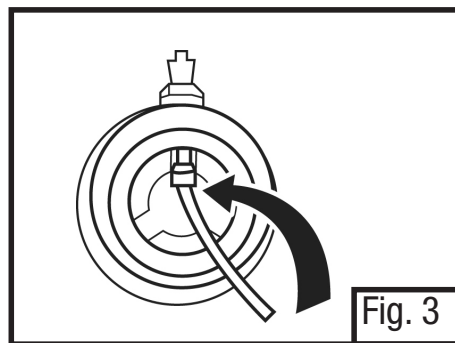
Fig. 3

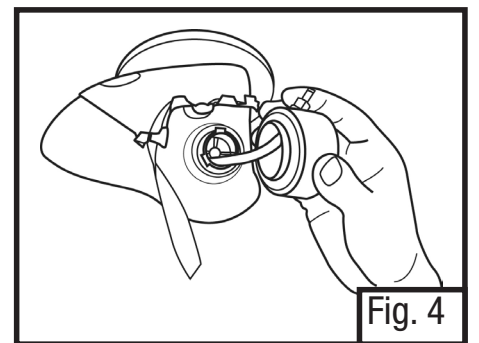
Fig. 4

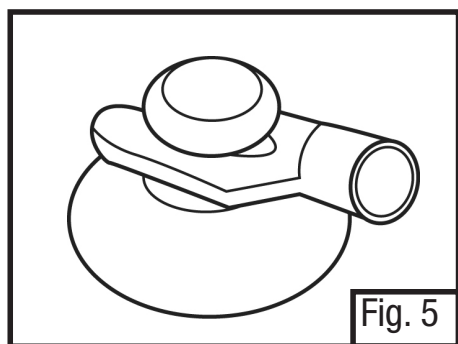
Fig. 5

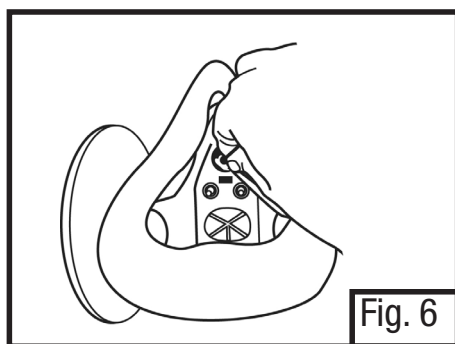
Fig. 6

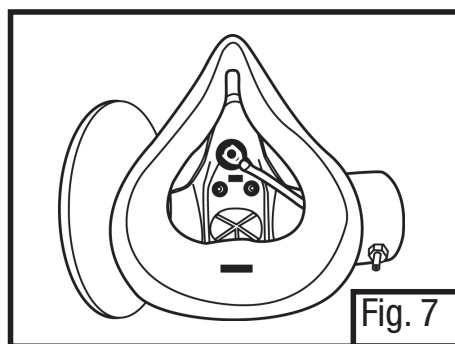
Fig. 7

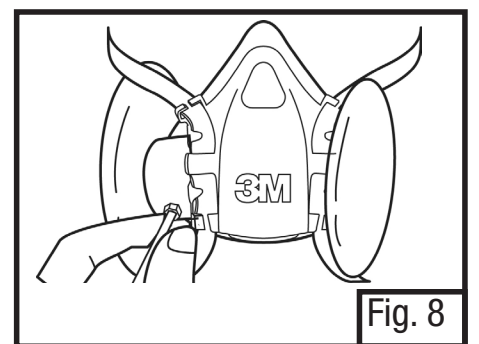
Fig. 8

3M Personal Safety Division

3M Center, Building 0235-02-W-70
St. Paul, MN 55144-1000

3M, 3M Scott and AV-3000 are trademarks of 3M or its affiliates,
used under license in Canada.

3M PSD products for occupational use only.

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⚠ WARNING

This product is to be used exclusively for quantitative fit testing. DO NOT use for respiratory protection. For correct use, consult supervisor or *User Instructions* or call 3M PSD Technical Service in U.S.A., 1-800-243-4630. In Canada, call Technical Service at 1-800-267-4414.

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10. Install a 3M™ P100 Filter onto the 601 Adapter. Be sure the same filter is also in place on the respirator's other filter holder.
11. Attach the quantitative fit test device's sample hose to the outside fitting of the 601 Adapter (Fig. 8).
12. Standard quantitative fit testing procedures can now be followed.

NOTE: Be sure to remove the 3M Quantitative Fit Test Adapter 601 assembly and make certain the respirator's inhalation valve is in place and functional before allowing the respirator to leave the fit test area.

FOR MORE INFORMATION

In United States, contact:

Website: www.3m.com/workersafety
Technical Assistance: 1-800-243-4630

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© 3M 2020.